



# NOBLE MANES CLINIC

SERVICES FOR BREEDERS BY BREEDERS  
Animal Clinic, Diagnostic Laboratory Testing, Breeder Support & Supplies



## Fecal Dx Profile with Giardia

### IDEXX TEST CODE

---

IDEXX 24639

### BREEDER TESTING OPTION

---

Single Animal or Combined Litter Sample

### IDEXX TEST TYPE

---

Immunoassays

### REQUIRED SUPPLIES

---

Collection Pack: Universal

### TURNAROUND TIME

---

1 day

### TESTS INCLUDED

---

Fecal Dx® Profile with Giardia

### SPECIMEN REQUIREMENTS: 24639

---

3 – 5 g (1 g minimum) fresh feces in a sterile container, RTFC (Red Top Fecal Container) preferred; keep refrigerated.

### STORAGE AND STABILITY

---

Fresh Feces up to 10 days at 36° to 46° F (2° to 8° C), keep refrigerated.

### TEST COMPONENTS: REFERRED

---

- Fecal Ova and Parasites
- Flea Tapeworm Antigen
- Giardia Antigen
- Hookworm Antigen
- Roundworm Antigen
- Whipworm Antigen

### INTERFERENCE

---

[Fecal O&P] poor sample quality / aged sample / small sample / freezing / non-formalin fixatives / samples contacting ground for >= 30 minutes. [Giardia] poor sample quality / aged sample / improper storage / PVA fixative / SAF.