

# RESCUE®

## WIPES

ONE STEP DISINFECTANT CLEANER & DEODORIZER

For Veterinary Use

## Reference Sheet

**KEEP OUT OF REACH  
OF CHILDREN**



Manufactured by:

**Virox Technologies Inc.**  
2770 Coventry Road  
Oakville, ON Canada L6H 6R1  
1-800-387-7578 | virox.com

EPA Est. No. 74559-CAN-1  
EPA Est. No. 075272-WI-001  
EPA Reg. No. 74559-10

This product is a one-step<sup>†</sup> disinfectant cleaner and deodorant designed for general cleaning and disinfecting of hard, non-porous surfaces.

It cleans quickly by removing dirt, grime, blood, urine, mold stains, laboratory stains, and other common soils found in animal housing facilities, grooming facilities, kennels, pet stores, veterinary clinics, or other small animal facilities.

### \*VIRUCIDAL: 1 MINUTE, UNLESS OTHERWISE NOTED

Adenovirus Type 8  
Avian Influenza A Virus (H7N9)  
Avian Influenza A Virus (H3N2)  
Avian Reovirus  
Bovine Viral Diarrhea Virus  
Canine Distemper Virus  
Canine Influenza A Virus (H3N2)  
Canine Parvovirus – 5 minutes  
Cytomegalovirus  
Duck Hepatitis B virus  
Equine Influenza A Virus (H3N8) (EIV)  
Feline Calicivirus  
Feline Herpes Virus (Feline Rhinotracheitis)  
Feline Infectious Peritonitis Virus (FIPV)  
Feline Leukemia Virus (FeLV)  
Feline Panleukopenia Virus  
Hepatitis B Virus (Duck Hepatitis B virus as the surrogate)  
Hepatitis C Virus (Bovine Viral Diarrhea virus as the surrogate)  
Herpes Simplex Virus Type 1  
Herpes Simplex Virus Type 2  
HIV Type 1 Strain HTLV III<sub>b</sub>  
Influenza A Virus  
Influenza B Virus  
Newcastle disease virus  
Norovirus (Feline Calicivirus as the surrogate)  
Pandemic Influenza A Virus H1N1  
Parainfluenza Virus Type 3  
Pseudorabies virus  
Respiratory Syncytial Virus (RSV)  
Rhinovirus Type-37  
Rotavirus (WA strain)  
SARS-Related Coronavirus 2 (Human Coronavirus SARS-CoV-2 (the virus that may cause COVID-19<sup>§</sup>))  
Vaccinia Virus

### BACTERICIDAL: 1 MINUTE

*Acinetobacter baumannii* (Multi-Drug Resistant)  
*Bordetella bronchiseptica*  
*Bordetella pertussis*  
*Burkholderia cepacia*  
*Enterobacter aerogenes*  
*Enterobacter cloacae* (New Delhi metallo-β-lactamase 1 (NDM -1))  
*Enterococcus faecalis* (Vancomycin Resistant (VRE))  
*Enterococcus faecium*  
*Escherichia coli*  
*Escherichia coli* O157:H7  
*Escherichia coli* (Extended Spectrum Beta-lactamase (ESBL))  
*Escherichia coli* (New Delhi metallo-β-lactamase 1 (NDM-1))  
*Klebsiella pneumoniae*  
*Klebsiella pneumoniae* (Extended Spectrum Beta-lactamase (ESBL))  
*Klebsiella pneumoniae* (New Delhi metallo-β-lactamase 1 (NDM-1))  
*Klebsiella pneumoniae* (Carbapenem Resistant (CRE))  
*Legionella pneumophila*  
*Listeria monocytogenes*  
*Pasteurella multocida*  
*Pseudomonas aeruginosa*  
*Pseudomonas aeruginosa* (Metallo-Beta lactamase (MBL) positive)  
*Salmonella enterica* (formerly known as *Salmonella choleraesuis*)  
*Serratia marcescens*  
*Shigella dysenteriae*  
*Staphylococcus aureus*  
*Staphylococcus aureus* Community Acquired Methicillin Resistant Genotype 400 (CA-MRSA 400)  
*Staphylococcus aureus* (Methicillin Resistant (MRSA))  
*Staphylococcus aureus* (Vancomycin Intermediate Resistant (VISA))  
*Staphylococcus aureus* (Vancomycin Resistant (VRSA))  
*Staphylococcus epidermidis* (Methicillin Resistant (MRSE))  
*Staphylococcus pseudintermedius*  
*Stenotrophomonas (Pseudomonas) maltophilia*  
*Streptococcus pneumoniae* (Penicillin Resistant (PRSP))  
*Streptococcus pyogenes*

### FUNGICIDAL: 30 SECONDS, UNLESS OTHERWISE NOTED

*Candida albicans* – 1 minute  
*Microsporium canis* – 1 minute  
*Trichophyton interdigitale*

### MALODORS

**Malodors** – Eliminates odors and odor-causing bacteria on hard, non-porous surfaces in restroom areas, behind and under sinks and counters, and storage areas and other places where bacterial growth can cause malodors.

## MOLD/MILDEW

**Mildewcidal Activity** – Kills the growth of mold and mildew at a 4 minute contact time: *Aspergillus brasiliensis* (formerly known as *Aspergillus niger*) and the odors caused by them when applied to hard, non-porous environmental surfaces.

**Mildewstatic Activity** – Controls and prevents the growth of mold and mildew at a 4 minute contact time: *Aspergillus brasiliensis* (formerly known as *Aspergillus niger*) and the odors caused by them when applied to hard, non-porous environmental surfaces.

## TUBERCULOCIDAL: 1 MINUTE

*Mycobacterium bovis* (BCG)

## NON-FOOD CONTACT SANITIZER: 10 SECONDS

*Acinetobacter baumannii*

*Campylobacter jejuni*

*Enterobacter aerogenes*

*Enterococcus faecalis* (Vancomycin Resistant (VRE))

*Escherichia coli* O157:H7

*Listeria monocytogenes*

*Pseudomonas aeruginosa*

*Salmonella enterica* (formerly known as *Salmonella choleraesuis*)

*Staphylococcus aureus*

*Staphylococcus aureus* (Methicillin Resistant (MRSA))

## DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Not for personal cleansing. This is not a baby wipe. Do not flush in toilet.

### Re-sealable Soft Pack

1. Remove the seal.
2. Remove one sheet at a time.
3. Close the seal to prevent drying between uses.

**\*KILLS HIV-1 AND HCV AND HBV ON PRE-CLEANED ENVIRONMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS** in health care settings and other settings in which there is an expected likelihood of soiling of (inanimate) hard non-porous surfaces / objects with blood / body fluids, and in which the surfaces / objects likely to be soiled with blood / body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS) Hepatitis B Virus (HBV), or Hepatitis C Virus (HCV).

### SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV OR HCV ON SURFACES / OBJECTS SOILED WITH BLOOD / BODY FLUIDS:

**Personal Protection:** Disposable latex or vinyl gloves, gowns, face masks, and eye coverings must be worn during all cleaning of body fluids, blood, and decontamination procedures.

**Cleaning Procedures:** Blood / body fluids must be thoroughly cleaned from surfaces and objects before application of this product.

**Contact Time:** Allow surface to remain visibly wet for 30 seconds to kill HIV-1, HBV and HCV.

**Disposal of Infectious Material:** Cleaning materials used that may contain blood / body fluids must be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

### For Use as a One-Step Cleaner/Disinfectant

1. Pre-clean visibly soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. Allow surface to remain visibly wet for 1 minute.<sup>8</sup>
4. Wipe surface dry or allow to air dry.

### To Kill Mold and Mildew

1. Pre-clean surfaces.
2. Pull towelette from canister and wipe onto hard, non-porous surfaces until completely wet.
3. Allow the surface to remain visibly wet for 4 minutes.
4. Allow to air dry.

### To Control Mold and Mildew

1. Pre-clean surfaces.
2. Pull towelette from canister and wipe onto hard, non-porous surfaces until completely wet.
3. Allow the surface to remain visibly wet for 4 minutes.
4. Allow to air dry.
5. Repeat application weekly or when growth reappears.

### For use as a Non-Food Contact Sanitizer

1. Pre-clean visibly soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. Allow the surface to remain visibly wet for 10 seconds.
4. Wipe surface dry or allow to air dry.
5. Change towelette after each use.

### For Use as a Cleaner and/or Deodorizer

Pull towelette from canister and wipe hard, non-porous environmental surfaces. Allow to air dry.

### Animal Housing Facilities in Veterinary Clinics and Hospitals

For cleaning and disinfecting the following hard non-porous surfaces: feeding and watering equipment, utensils, instruments, cages, kennels, catteries, etc.

1. Remove all animals and feed from premises, animal transportation vehicles, crates, etc.
2. Remove all litter, droppings and manure from floors, walls and surfaces of facilities occupied or traversed by animals.
3. Empty all troughs, racks, and other feeding and watering appliances.
4. Thoroughly clean all surfaces with soap or detergent and rinse with potable water. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
5. All surfaces must remain visibly wet for 1 minute.<sup>8</sup>
6. Allow to air dry.
7. Thoroughly scrub all treated feed racks, mangers, troughs, automatic feeders, fountains, and waterers with soap or detergent, and rinse with potable water before reuse.

## STORAGE AND DISPOSAL

**PESTICIDE STORAGE:** Store this product in a cool dry area, away from direct sunlight and heat. Do not contaminate water, food or feed by storage and disposal.

**CONTAINER HANDLING:** Dispose of wipe in trash after use. Do not flush. Tightly close lid/flap between uses to retain moisture. Non-refillable container/dispenser.

Do not reuse or refill this container/dispenser. Offer empty container/dispenser for recycling. If recycling is not available, discard container/dispenser in trash.

<sup>†</sup> Accelerated Hydrogen Peroxide® and Design are registered trademarks of Diversey Inc. | "Accelerated" does not imply increased product efficacy. | For relevant patents, visit: [virox.com/patents](http://virox.com/patents).

<sup>†</sup> When used according to disinfection use directions

<sup>8</sup> Use a 5 minute contact time against Canine Parvovirus.

<sup>§</sup> On hard, non-porous surfaces.